

APEX®

ICD™

IRREGULAR CORNEAL DESIGN

FLEXFIT

Diagnostic Selection

The APEX ICD FlexFit Scleral design enables you to have complete control over the lens allowing for comprehensive customization to meet your individual patient's needs.

Available in two options:

The larger 16.3mm diameter is for highly asymmetric, diseased and irregular corneas. The diameter can be modified within the range of 15.5 to 17.0 and each zone can be extensively modified to customize the construction for virtually every eye.

The smaller 14.8mm diameter is for regular eye shapes, smaller corneas and less asymmetric conditions. The diameter can be modified within the range is 14.5 to 15.5 and each zone can be extensively modified to customize the construction for virtually every eye.

Contact your Account Manager for pricing:

Amy Chow: Eastern Canada
Cardinal Contact Lens
amy@cardinalcontactlens.com
437 993 2020

Trevor Kojima: Western Canada
Precision Technology Services
trevor@ptsoptics.com
604-215-4206

Condition	DIA.	SAG	BC	Diop	Power
Normal Based on this Corneal Condition, Select the 4000µm Trial Lens to start	16.3 DD	3800	8.45	40.00	+2.00
	16.3 DD	4000	8.25	41.00	Plano
Keratoconus Based on this Corneal Condition, Select the 4400µm Trial Lens to start	16.3 DD	4200	8.05	42.00	-2.00
	16.3 DD	4400	7.85	43.00	-4.00
	16.3 DD	4600	7.50	45.00	-6.00
Transplant Based on this Corneal Condition, Select the 4800µm Trial Lens to start	16.3 DD	4800	6.90	49.00	-8.00
	16.3 DD	5000	6.60	51.00	-10.00
	16.3 DD	5200	6.25	54.00	-12.00
DD = Dual Depth	16.3 DD	5400	6.25	54.00	-14.00

Condition	DIA.	SAG	BC	Diop	Power
Normal Based on this Corneal Condition, Select the 3400µm Trial Lens to start	14.8	3400	8.45	40.00	-4.00
	14.8	3600	7.85	43.00	-6.00
Keratoconus Based on this Corneal Condition, Select the 3800µm Trial Lens to start	14.8	3800	7.50	45.00	-8.00
	14.8	4000	7.20	47.00	-10.00
Transplant Based on this Corneal Condition, Select the 4200µm Trial Lens to start	14.8	4200	6.90	49.00	-12.00

Your Partners in Visual Excellence.

pts PRECISION
TECHNOLOGY SERVICES

Cardinal
Contact Lens Inc.

Western Canada
Precision Technology Services Ltd
866 Cordova Street East, Vancouver,
BC, Canada V6A 1M4
Phone: **604 215 1467**
Toll Free: **1 800 663 4248**
Email: **sales@ptsoptics.com**

Eastern Canada
Cardinal Contact Lens Inc.
#7-38 Cowansview Rd., Cambridge,
Ontario, Canada N1R 7N3
Tel: **519 740 3050** • Fax: **519 740 3296**
Toll Free: **1 800 263 3973**
E-mail: **info@cardinalcontactlens.com**